

## DryMax Super

## DryMax Blue

### Instructions for Use

#### Indications:

DryMax Super/Blue are sterile superabsorbent wound dressings for use on exuding wounds.

#### Product properties:

These wound dressings have a superabsorbent core which absorbs and retains exudate. The exudate is absorbed vertically by the dressing from the white wound contact side and is retained in the core forming a gel, thus minimizing the risk of maceration to the peri-wound skin while maintaining a moist wound environment. DryMax Blue has a hydrophobic coloured backing layer (side up) to minimize moisture strike-through.

#### Instructions for use:

The dressings should be used on exuding wounds (patients from 1 year and older) and handled by healthcare professionals. The dressings can be used as either primary or as secondary dressings. Clean the wound as per local protocol. Dry the surrounding skin carefully. Choose a size of dressing depending on the wound size; the core edge of the dressing should overlap the wound with a 2 cm margin as the sealing edges do not absorb exudate. Apply the dressing with the white side in direct contact with the wound bed or the primary dressing. Secure the product in place. The dressings can be used in combination with compression therapy. The dressings should be changed according to the clinical condition of the wound, or before saturation. If the wound deteriorates, consult your healthcare professional. Dressing removal can be facilitated by moistening the dressing. Dispose of the dressing as clinical waste according to local regulations. In case a serious adverse event has occurred in relation to usage of any of the devices, this should be reported to Absorbest AB and the competent authority in the country in question.

#### Caution:

Do not cut the dressing, the cover must remain intact. Do not use the dressing if the sterile packaging is damaged. Single use product, do not re-sterilize, can not be re-used. Important to use the white wound contact side towards the wound!

#### Contraindications:

DryMax wound dressings should not be used on eyes, mucous membranes, tendons, on dry wounds or in wound cavities because the dressing swells during absorption. DryMax wound dressings should not be used on persons with a known sensitivity to the dressing or its components.




Do not reuse. Reusing will cause cross contamination.



Do not use if the sterile barrier or packaging is compromised.



Do not cut.

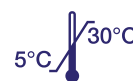

Absorbest AB  
Klintvägen 1  
SE-590 39 Kisa  
Sweden  
[info@absorbest.se](mailto:info@absorbest.se)  
[www.absorbest.com](http://www.absorbest.com)

