

DryMax Non Sterile


DryMax Non Sterile is a conformable, superabsorbent dressing for exuding wounds. Its properties provide an easy-to-use and reliable solution for both patient and clinician. DryMax Non Sterile absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core, forming a gel, thus minimizing the risk of maceration and maintaining a moist and balanced wound environment.

DryMax Non Sterile should be used as a secondary dressing. The seamless cover simplifies positioning over the wound. The design allows the dressing to be applied with either side facing the wound. This feature also makes it possible to stack dressings on each other if extra absorption capacity is required.


The dressing remains dry and comfortable for the patient. Simultaneously, it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the dressing's core constructed by our DryMax Technology.

The ultra-thin and highly conformable features allow the dressing to be used on all body parts. DryMax Non Sterile is designed to work well together with compression therapy.


Application Guide

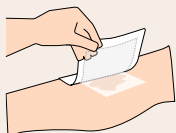

The dressing must be placed in close combination with a suitable primary dressing.

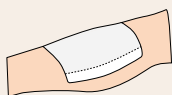

Secure the dressing with suitable bandage, film or tape.

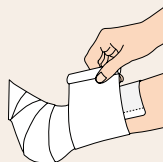

DryMax Non Sterile can be used in conjunction with compression therapy.


Item	Product size cm	REF	Pcs/box	Absorption-capacity*
DryMax Non Sterile	10x10	F20011/200	200	50 ml
DryMax Non Sterile	10x22	F20021/100	100	150 ml
DryMax Non Sterile	20x22	F20041/100	100	300 ml
DryMax Non Sterile	20x30	F20051/75	75	400 ml
DryMax Non Sterile	37x56	F20071/10	10	1600 ml

*According to international standard EN 13726



- **Cost effective**
- **User friendly**
- **Increased patient comfort**
- **Improved wound healing***

*Compared with conventional wound dressings

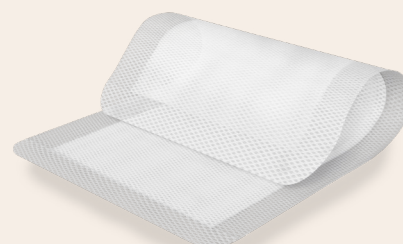

DryMax® technology